**IPOS Medical**

EU Distributor: **IPOS - HANDEL GmbH**  
 Robert-Bosch-Str. 1 | 89250 Senden | +49 7207 35904-33  
 info@ipos-medical.de | www.ipos-medical.de

**(EU) 2016/425**
**CE 2841**

**EN 149:2001+A1:2009 FFP2 NR**

**ATEMSCHUTZMASKE FFP2**

(EN) Protective Mask FFP2

(ES) Mascarilla de respiración FFP2

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2022/01/01

**PROTECTIVE MASK FFP2 (EN)**

**Purpose**  
 The FFP2 mask is personal protective equipment (PPE) and primarily protects the wearer of the mask from particles, droplets and aerosols. It serves to protect oneself and others by filtering the air when inhaling and exhaling.

**Putting on the mask**  
 Please observe the pictograms that describe how to put on the mask in these instructions and on the packaging.

**Before use**  
 1. Ensure that your hands are clean before putting on the mask. Please check all the individual parts of the mask for damage. If the mask is damaged, do not use it; instead, dispose of it immediately.

- Wash your hands thoroughly before putting on the mask.
- Unfold the mask into a shell shape.

- Place the mask over the nose and mouth area and adjust the nose clip to the shape of your nose.
- Put the ear straps over both ears with the opposite hands.

6. Check that the mask is in the correct position before use by checking the fit as follows:

- Cover the mask with both hands without compromising its position.
- Exhale strongly. If air escapes around your nose, adjust the shape of the nose clip as described above.
- If air escapes at the edges of the mask, adjust the position of the ear straps. You may not enter the danger zone if your mask is not positioned correctly.

**Warnings and limitations**

- The user should familiarize themselves with the respiratory protective mask before use and observe the instructions for use.
- Check the mask for any signs of damage before use.
- Facial hair below the sealing band on the face can cause air to escape.
- The mask must not be squeezed or deformed.
- Do not wash.
- Do not place a towel, handkerchief, etc. inside the mask.
- Do not use if the mask is contaminated.
- Do not touch the surface of the mask after you have worn it.
- Store the mask according to the storage conditions specified on the packaging.
- **THE WARNINGS DESCRIBED HERE APPLY TO THE ENTIRE TIME YOU ARE USING THE MASK. FAILURE TO OBSERVE THESE WARNINGS MAY LEAD TO DISEASES AND HARM TO HEALTH**

**Approvals**

This product is approved according to DIN EN 149:2001 + A1:2009 FFP2 NR.

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